



# CPAP cleaning should be simple.

Introducing the faster, easier way to clean CPAP equipment. SoClean kills 99.9% of CPAP germs and bacteria in your mask, hose and reservoir.

**SoClean.**



“I actually use my CPAP every night now because it’s always clean!”

Bryon Farnsworth

## Now, CPAP cleaning is a snap.

After a simple one-time setup, the SoClean sanitizes and disinfects the CPAP mask, hose and reservoir - no disassembly, water or messy chemicals required. Your CPAP equipment remains completely dry.

The SoClean makes it easy for any CPAP user to safely and naturally clean and sanitize their equipment on a daily basis. It quickly kills 99.9% of common CPAP germs and bacteria. Your CPAP will be ready for use so you can effortlessly get a better night’s sleep.



# #SoCleanSoEasy

## NO DISASSEMBLY

Sanitize all of your CPAP equipment without taking apart the CPAP mask, hose, or reservoir.



## KILLS CPAP GERMS

Kills 99.9% of CPAP germs, and bacteria safely and naturally with no water or chemicals needed!



## SAVE TIME

CPAP cleaning shouldn't take you away from your day. The automated SoClean does the work for you.





## The Faster, Easier, More Effective Solution


For individuals seeking a better way to maintain a clean CPAP, the SoClean Automated CPAP Equipment Sanitizer is a highly effective solution – plus it's faster and easier than other cleaning methods.

The SoClean uses ozone – also known as O<sub>3</sub> or activated oxygen – to sanitize CPAP equipment. Ozone is a 100% safe, naturally occurring gas that has been used to purify water since the 1800s. In today's world, it's a trusted sanitizing process used by hospitals, food handlers and the hotel industry.

Ozone has been shown to be effective in destroying dozens of harmful pathogens, including E. coli, Influenza Virus, Staphylococci, Streptococcus Bacteria, Stomatitis Virus and many more.

In addition to being highly effective, the SoClean is also easy-to-use. After a simple, one-time setup, CPAP users simply place their mask in the SoClean, close the lid and walk away. No disassembly of the mask, hose or water reservoir is required – and no soap and water, wipes or messy chemicals are needed. When the SoClean is done sanitizing the user's CPAP equipment, it will be completely dry and ready to use.

# How does SoClean Work?

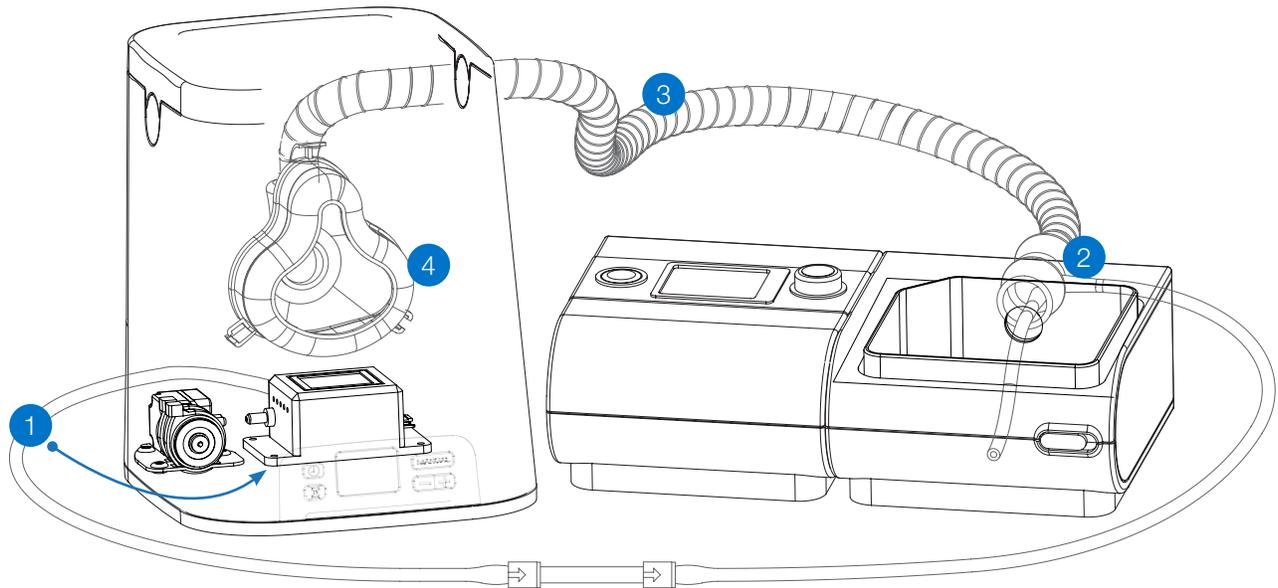

## 1. Generate Ozone

The SoClean generates ozone ( $O_3$ ) and it is pumped through the SoClean injection hose.

## 2. Sanitizes the Reservoir & Water

The injection fitting connects your SoClean with the CPAP reservoir. Ozone saturates the entire reservoir – sanitizing everything including the water.

## 3. Sanitizes the Hose

Ozone is a gas and floats up and through the hose - sanitizing all the crevices.

## 4. Sanitizes In & Outside of the Mask

Ozone surrounds the CPAP mask – sanitizing the inside, outside and headgear!

Did you know that SoClean uses the same sanitizing process found in:



Water  
Purification




Produce  
Handling



Hotel  
Housekeeping



Hospital  
Sanitizing

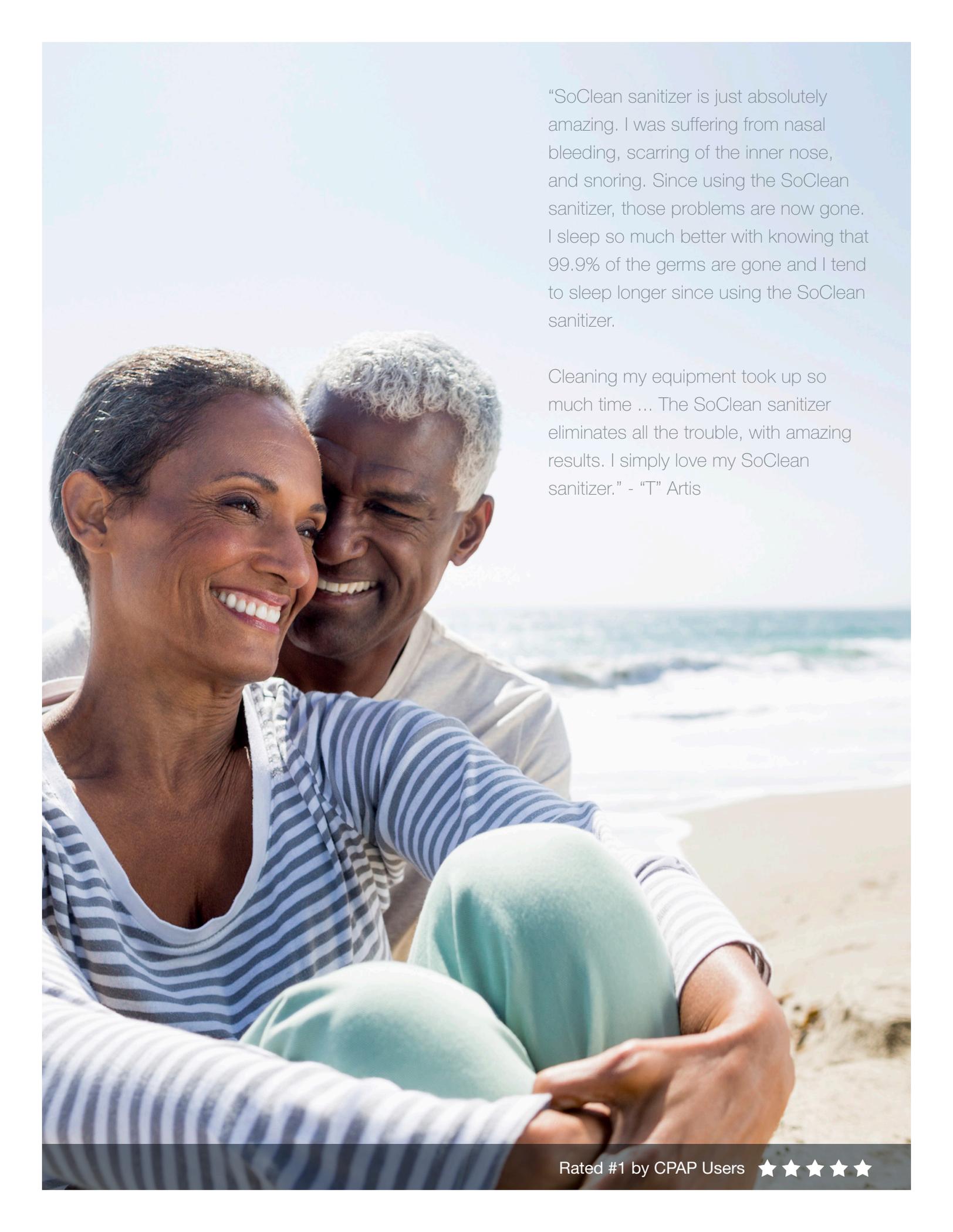A photograph of a middle-aged couple sitting on a sandy beach. The woman, in the foreground, is wearing a blue and white striped long-sleeved shirt and light blue pants. She is smiling broadly, looking towards the man. The man, with white hair, is wearing a white t-shirt and is also smiling, looking towards the woman. They are sitting close together, with the man's arm around the woman. The background shows the ocean with waves breaking on the shore under a clear blue sky.

“SoClean sanitizer is just absolutely amazing. I was suffering from nasal bleeding, scarring of the inner nose, and snoring. Since using the SoClean sanitizer, those problems are now gone. I sleep so much better with knowing that 99.9% of the germs are gone and I tend to sleep longer since using the SoClean sanitizer.

Cleaning my equipment took up so much time ... The SoClean sanitizer eliminates all the trouble, with amazing results. I simply love my SoClean sanitizer.” - “T” Artis

Rated #1 by CPAP Users ★★★★★

“If you’re using a CPAP machine, you could develop a respiratory illness if it’s not cleaned properly. That’s why I strongly recommend SoClean - the world’s first automated CPAP cleaner and sanitizer. SoClean kills 99.9% of CPAP germs and bacteria, without hazardous chemicals and it works on all popular CPAPs.”

Joseph Krainin, M.D., FAASM  
Founder and President of Singular Sleep, LLC

A CPAP is essential to treating obstructive sleep apnea. It should only be making obstructive sleep apnea patients feel better – not worse. The best way to prevent pneumonia, the flu and other infectious diseases is to keep one’s CPAP clean, but some of the recommended cleaning methods just aren’t cutting it.

Cleaning should be simple. That’s the philosophy behind the SoClean Automated CPAP Equipment Sanitizer. To learn more about this #1-rated CPAP-cleaning device, visit [www.SoClean.com](http://www.SoClean.com).





Sanitize your  
CPAP equipment  
hands-free!

## Contact

---

SoClean Inc.  
12 Vose Farm Road  
Peterborough, NH 03458

Sales (800) 747.1030  
Support (800) 341.7014  
Intl +1(508) 363.0415



[info@SoClean.com](mailto:info@SoClean.com)

[www.SoClean.com](http://www.SoClean.com)

© 2018 SoClean® is a registered trademark owned by SoClean Inc. All Rights Reserved.  
U.S. Patent No. 9,358,316 | 9,610,373 | 9,616,147 | 9,669,124 | D719,674 | D719,673.